Lack B eco

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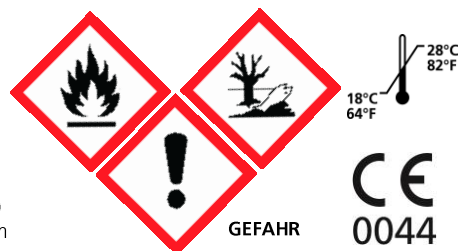
Directions for use

Product Description:

A silicone lacquer for silicone earmolds (Biopor®) or all silicone impressions (also made of Otoform®) which cures in the air; also suited as an adhesive coating for acrylic earmolds (treat with Glusil in advance).

The cured coating has a brilliantly shiny and clear-transparent appearance, the surface is easy to clean. The bond between the silicone earmold and the layer of the Lacquer B eco is excellent, the coating is very durable and long lasting.

Processing:

Apply Lacquer B eco with a brush onto the clean earmold (wipe off grease with a moist leather cloth) or dip the whole earmold into it. For an adhesive coating apply Glusil (REF 076) on the acrylic earmold. Curing time: approx. 10 minutes, then treat with Lacquer B eco, curing time in the air: approx. 15 - 20 minutes. Please use a suitable suction device during the processing.

Curing:

Exposed to air; approx. 15 - 20 minutes (at 23 °C ± 1 °C, 50 % - 60 % RH)

Storage:

Keep bottle well-sealed at room temperature

Dilution:

with diluting agent for Lacquer B eco and Lacquer B mat (REF 0817)

REF

0811 - 20 ml bottle / **0812** - 50 ml bottle / **0813** - 250 ml bottle

Attention:

Insert a just lacquered earmold into the ear not before 30 minutes!

Low temperatures and low humidity delay the curing time of the lacquer!

Possible dangers:

Before use observe the Material Safety Data Sheet of Lacquer B eco.

Hazard statements

H225 Highly flammable liquid and vapour.
H315 Causes skin irritation.
H319 Causes serious eye irritation.
H410 Very toxic to aquatic life with long lasting effects.

Precautionary statements

P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.
P241 Use explosion-proof electrical/ventilating/lighting/equipment.
P280 Wear protective gloves/protective clothing/eye protection/face protection.
P303+P361+P353 IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water/shower.
P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
P501 Dispose of contents/container in accordance with local / regional / national/international regulations.

Contains Diocetylzinnacetylacetonate. May produce an allergic reaction.

Lot number / Best before date: The lot number and the best before date are indicated on each Lack B eco packaging. In case of claims please always indicate the lot number of the product. Do not use the product after expiry of the best before date.

Dreve Otoplastik GmbH is not liable for any damages caused by improper application of the material.

2020-01-10 / Rev. 2
Dreve Otoplastik GmbH
Max-Planck-Str. 31 • 59423 Unna / Germany
Tel.: + 49 2303 8807-0 • Fax: + 49 2303 82909
E-Mail: otoplastik@dreve.de • Internet: www.dreve.com