

Cat. #101

Huggie Aids for Larger Hearing Aids

Huggie Aids are made in 1/4" increments packaged in pairs only. Huggie Aids will give you an exact fit if you measure ear correctly; see below.




1. Measure around the ear with string, as in the picture; make string snug but do not distort the ear lobe. Measure length to the nearest 1/4". Should 1/8" be measured, go up to the next 1/4", eg. 4 1/8", go to 4 1/4". Huggie Aids are very flexible for a perfect fit. The straps are strong for long endurance. Do not leave



Huggie Aids on your patient so long that the ear lobe distorts; tell parent to watch for the ear to become too big for Huggie Aid and return for a new one.

2. Remove Earmold; slip hook end into 'A' strap keeping the tube under the hearing aid. Slide 'B' strap over the bottom of the hearing aid; then cover battery door, keeping the tube portion of Huggie Aid to underside of the hearing aid close to the ear. Push mold into the ear through the Huggie Aid circle, not under it.

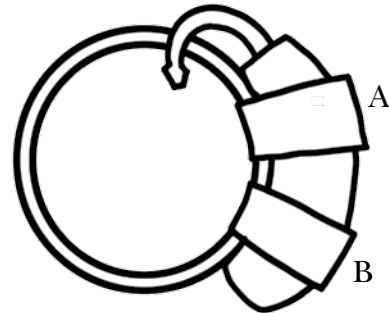

When cartilage is too soft for a Huggie Aid, ask about the Huggie Head Cat. #1105 designed for monaural or binaural BTE hearing aids.

Patent No. 4702345