Härterpaste

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Produktbeschreibung

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Directions for use

Product Description

Hardener paste for condensation vulcanizing impression materials for ear impression taking. Colored for a mixing control when mixing with the impression material.

Indication for use:

Hardener paste for ear impression material intended to reproduce a patients external auditory canal.

Processing

Add hardening paste according to the quantity instruction of the respective impression material (approx. 2 resp. 4 %). Mix until an equal colour is obtained.

Storage

18 –28 °C, Keep tube well closed.

Item-No.

	Color
	red
5 ml tube	52112
20 ml tube	52212
100 ml tube	52412

Attention

Hardener paste contains organic tin compounds and may irritate the skin. The direct skin contact with the hardener paste has to be avoided as the paste is irritating.

Hazard statements

- H315 Causes skin irritation.
H317 May cause an allergic skin reaction.
H373 May cause damage to organs through prolonged or repeated exposure.

Precautionary statements

- P260 Do not breathe dust/fume/gas/mist/vapours/spray.
P280 Wear protective gloves/protective clothing/eye protection/face protection.
P308+P313 IF exposed or concerned: Get medical advice/attention.
P362 Take off contaminated clothing.
P363 Wash contaminated clothing before reuse.
P501 Dispose of contents/container in accordance with local/regional/national/international regulations.

Contains: Dioctyltinacetylacetonate, dioctyltin oxide

Dreve Otoplastik GmbH is not liable for any damages caused by improper application of the material.

2019-06-27 / Rev. 1
Dreve Otoplastik GmbH
Max-Planck-Str. 31 • 59423 Unna / Germany
Tel.: + 49 2303 8807-0 • www.dreve.com

Warning